



Zimmer Cryo 6


Filter Replacement Instructions

Instructional videos also available for viewing on Zimmer USA's YouTube Channel (www.youtube.com/ZimmerUSA)

How to replace your Zimmer Cryo 6 Filter

1. Unplug the power cord from the power source
2. Remove the two rear wheel covers in the back of the unit. (With two fingers, grab underneath the wheel. Push down with your thumbs and pull toward you.)
3. Remove the four screws from the bottom plate on the back of the unit
4. Remove bottom plate
5. Remove filter and remove safety ground connector from filter
6. Attach ground connector to new filter
7. Reinstall new filter and pull it up
8. Reinstall the bottom plate with four screws
9. Slide the two rear wheel covers back into place
10. Plug power cord back into power source

Cryo 6 Filter Replacement Instructional Video:
<https://youtu.be/t5rpEa8JF8Y>

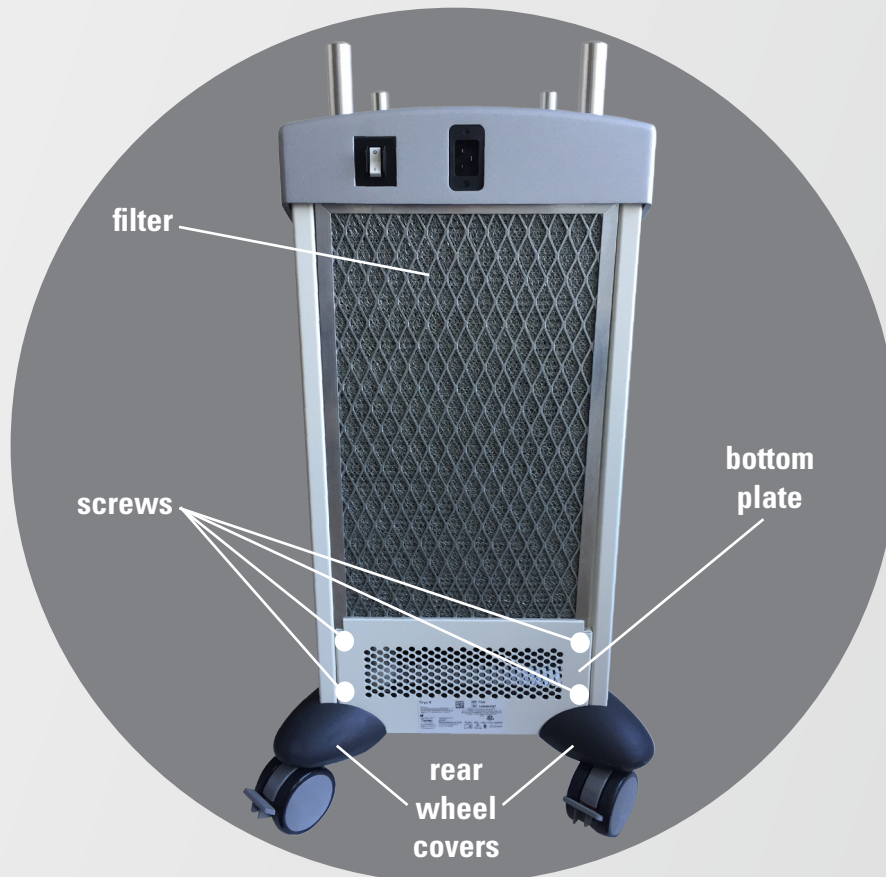

For Zimmer Parts and Service



800-327-3576



service@zimmerusa.com



shop.zimmerusa.com